G19UHS02

IT

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EN

**INSTRUCTIONS AND
PRECAUTIONS FOR SAFE
USE.**

Prior to installing and using the device, carefully read the instructions for use and – in particular – the safety precautions, and comply with them. Store this manual for reference for the entire life-cycle of the device. Should this manual get damaged, or if certain sections are difficult to understand or give rise to doubts, prior to using the device contact the company at the address indicated on the last page. Any changes to the information provided in the instruction manual – if deemed important for the user's safety – shall be announced through the Ponte Giulio Internet website and notified to all authorised dealers.

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EN

GENERAL INFORMATION

The device is safe to use, even for people with reduced visual, reading or cognitive abilities. There are no circumstances hampering ease of use of the product. The device must not be dismantled and does not have dismantable parts. The device's overall weight does not exceed 10 kg. The device does not contain

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carcinogenic, mutagenic or toxic substances.
It is capable of withstanding corrosion and temperature variations.

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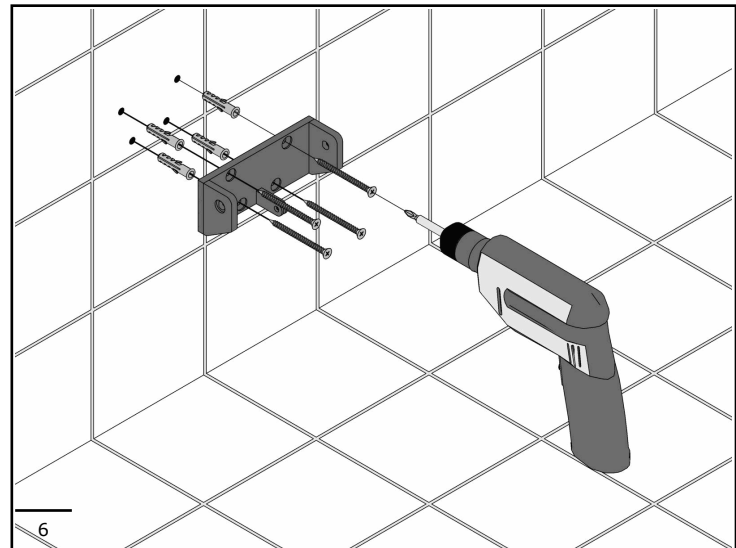
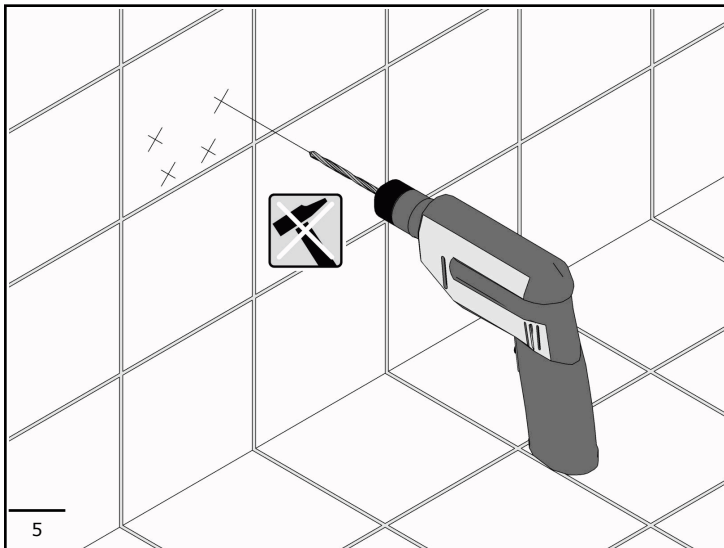
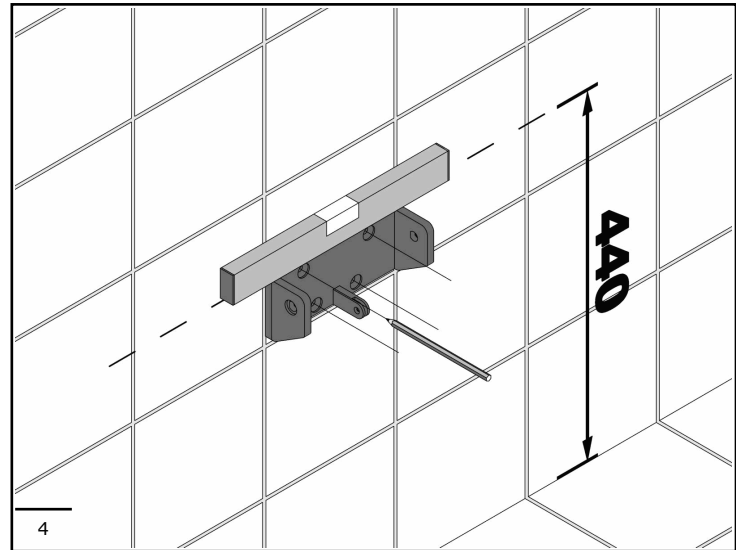
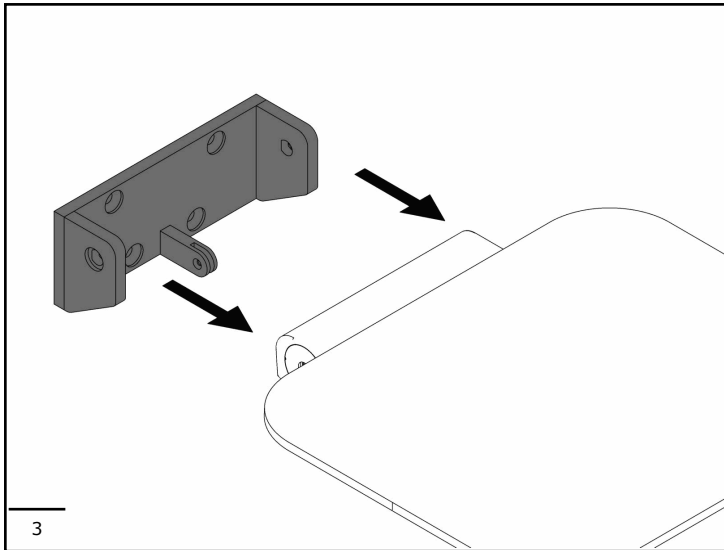
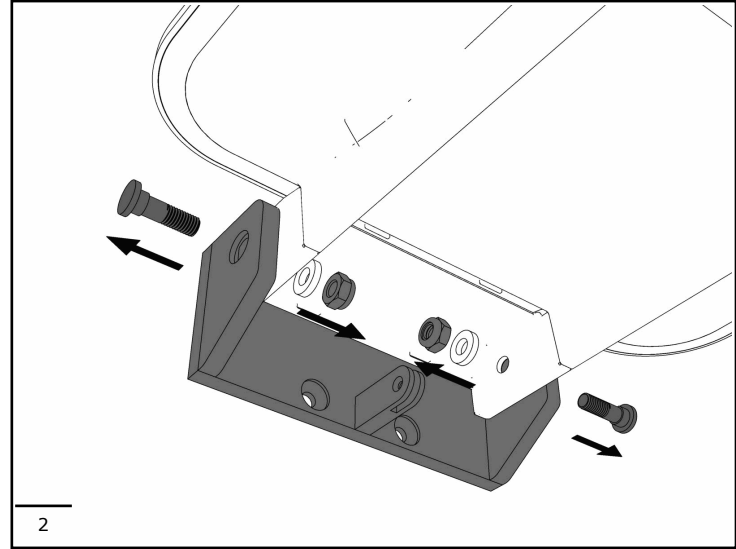
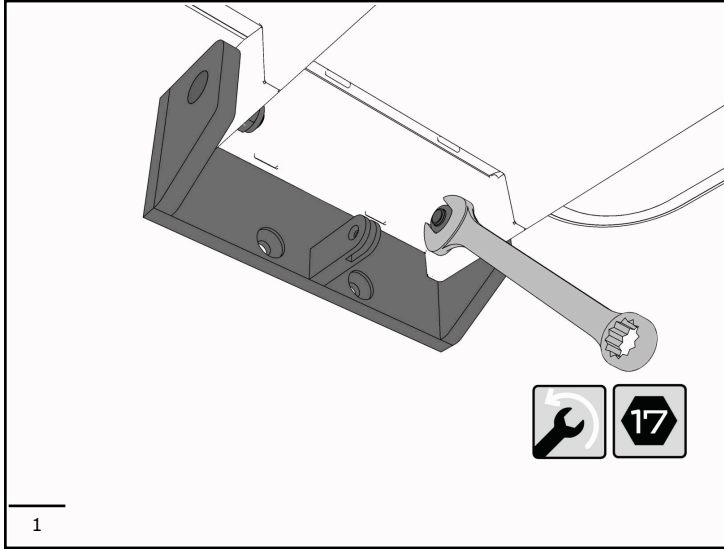
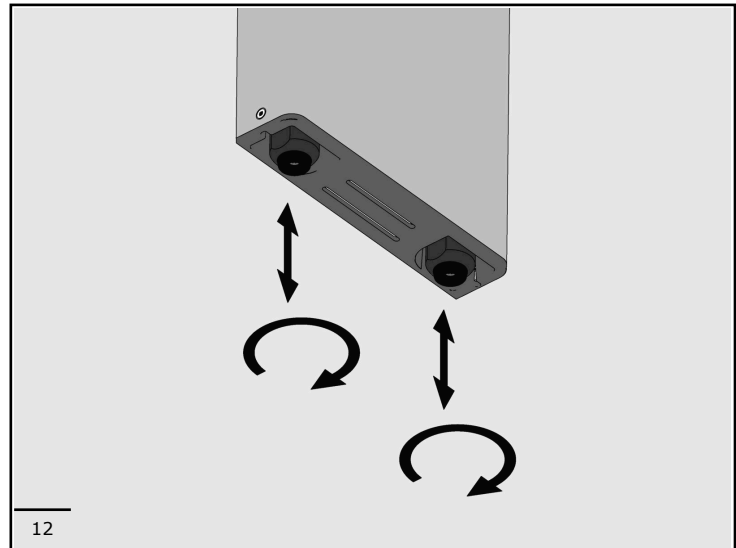
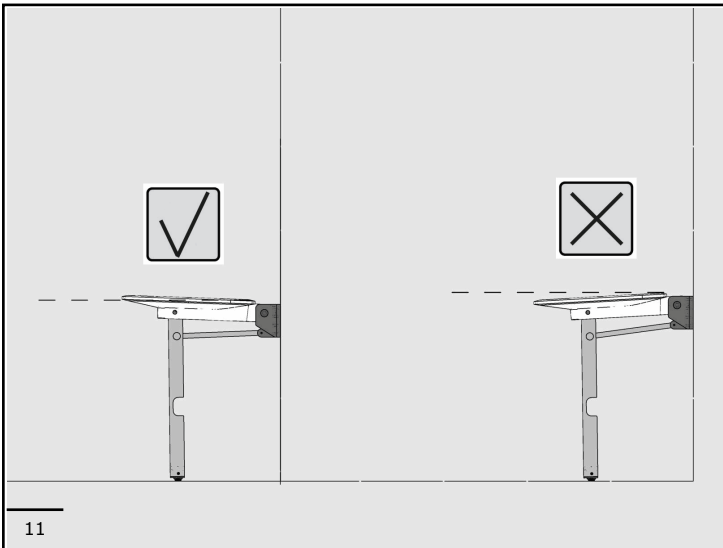
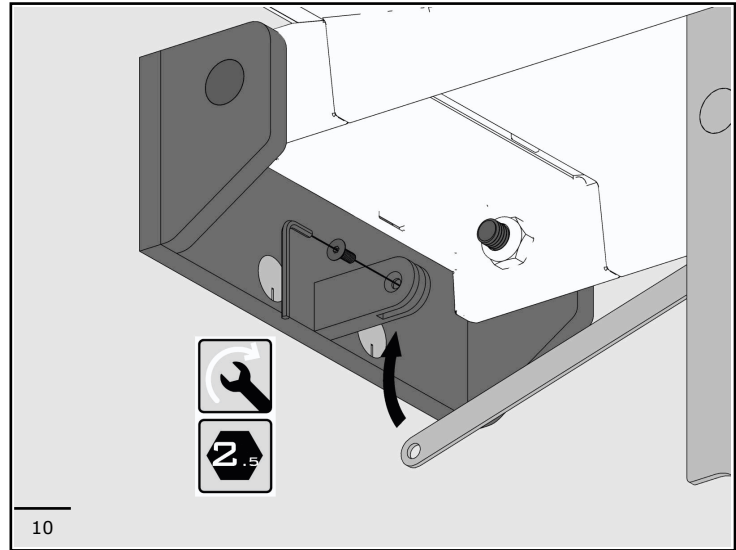
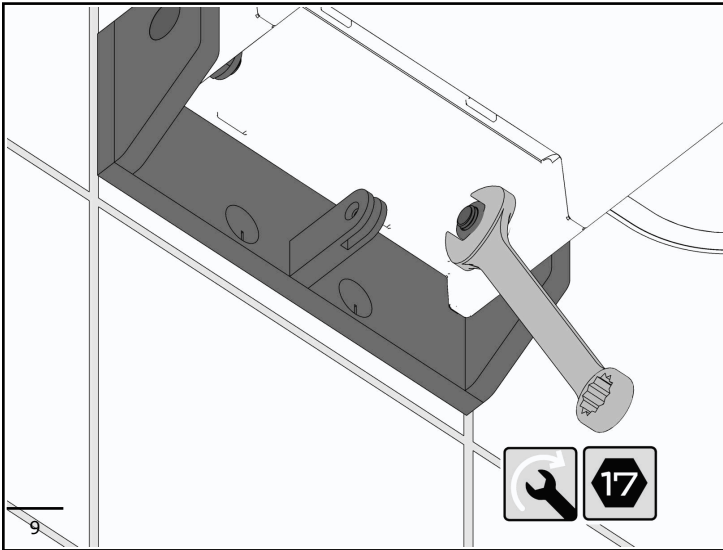
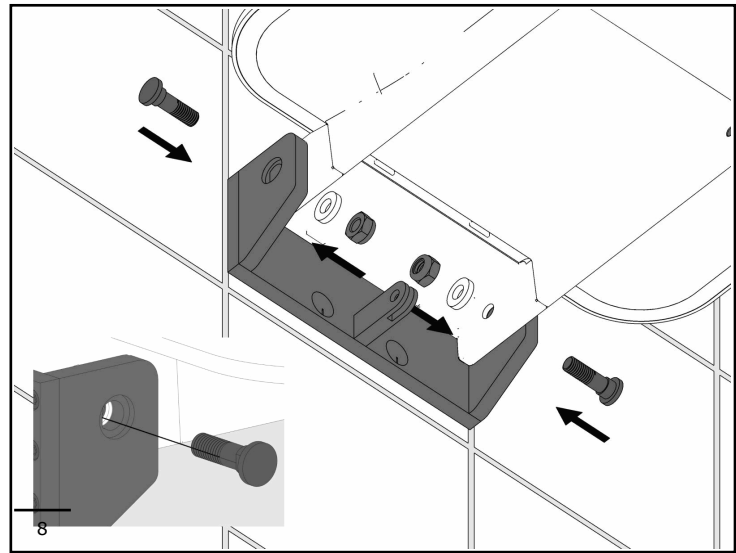
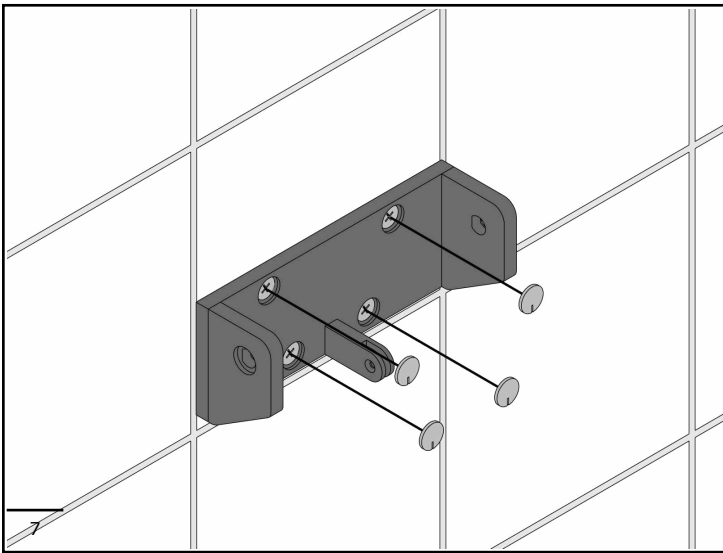
EN

INSTRUCTIONS FOR USE

The device is a seat exclusively for sitting. Using the device improperly, for example for gymnastics exercises, can cause injuries with serious

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consequences.
Do not subject the device to loads exceeding for which it was designed.
Exceeding this load can cause permanent damage to the device or harm to people.
To ensure correct use of the device, the latter must be left free of any objects and be easily reachable.

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The device must be installed by trained and qualified personnel (bricklayer or professional plumber) who must verify whether the fixing elements, if provided, are suitable for the relevant wall.
To guarantee stability and safety, the device must be attached with wall plugs and screws suitable for the relevant type of wall.
Do not subject the device to loads exceeding those specified.
If the device should become unstable, suspend its use until it is properly checked by the relevant technician.
The manufacturer declines all responsibility for installation of the device not conforming in this manual for use and maintenance.

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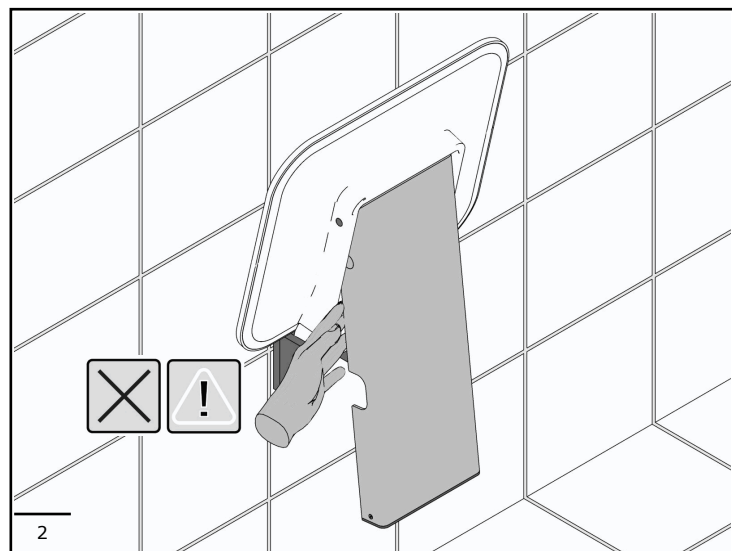
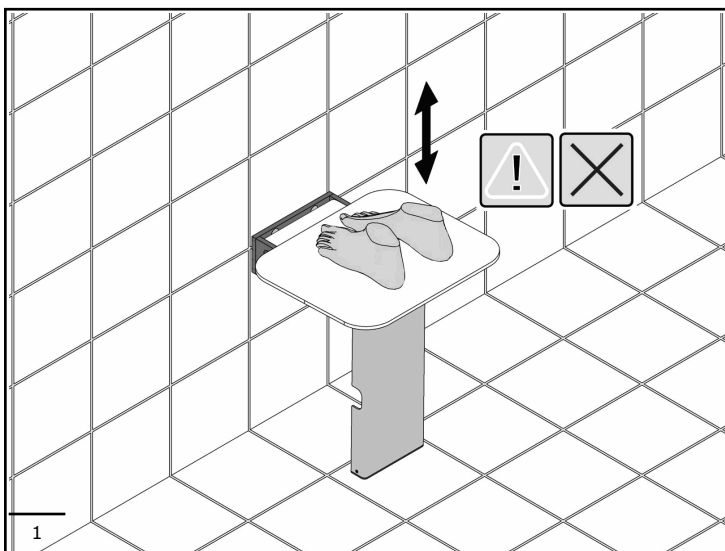
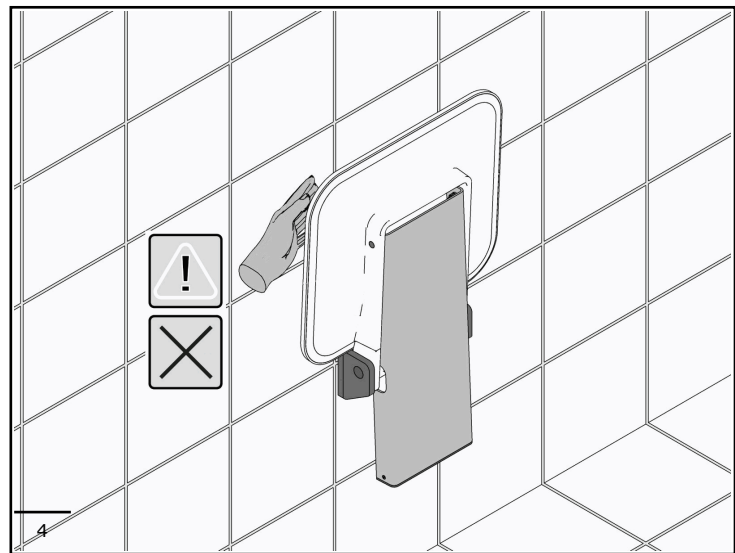
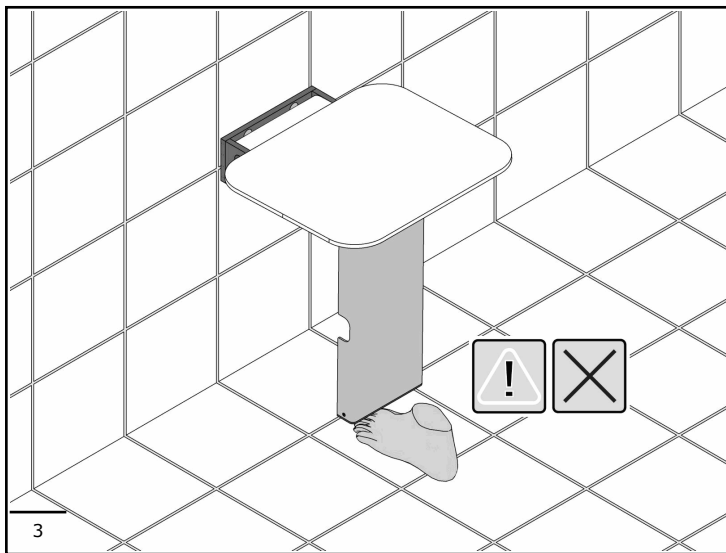
EN

CLEANING

Clean the surface of the device using a damp cloth wetted with liquid neutral soap (e.g. dish soap). It is forbidden to use abrasive liquid detergents containing chlorine, strong acids or bases, concentrated solvents such as acetone and trichloroethylene or derivatives, which may cause the material too dull. It is forbidden to use hot air sources for drying. Disinfect the device's surface using a bleach-type product at least once a week.

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EN

MAINTENANCE

Check the stability of the wall-mount at least every 3 months. Should the device become unstable, contact the installer. Check whether the device pivots, at least every 3 months. If it pivots freely, tighten the pin until the tilting is allowed only by user intervention. The device shall be fully replaced in case of any damages.

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IT

The device is not affected by the room's temperature, though it must NOT be installed in environments subject to temperatures below 4°C or above 40°C.
Do not install in saunas. Not comply with the indicated temperatures may damage the plastic coating of the device.

EN

DISPOSAL

The user is responsible for the device's disposal, which must occur in accordance with the laws of the country in which the device is installed. The device is not manufactured using substances that are hazardous to human health and the environment, and all parts comprising it are recyclable and/or reusable. In order to safeguard the environment, the device also contains a certain amount of recycled materials.
Keep away from children packaging; risk of suffocation.

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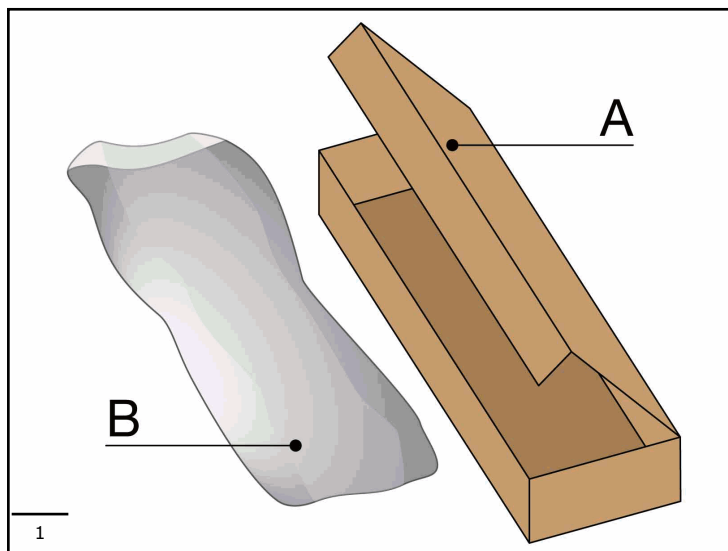
EN

A
 Packaging type: Box
 Coding: PAP 20 - Corrugated board
 Collection instructions: PAPER

B
 Packaging type: Sack
 Coding: LDPE 4 - Plastic
 Collection guideline: PLASTIC

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EN

FURTHER INFORMATION

For further information, please visit our website or contact the our dealer zone

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EN

NOTE

Ponte Giulio reserves the right to make changes to the products or to their specifications without prior approval or notification to the customer for:

- Changes required for safety purposes,
- Changes that do not adversely affect the product performance,
- Changes needed to meet the product specifications,
- Changes needed to comply with legal requirements or applicable regulations

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